

# Instruction for use

## Microsurgical Knife (sterile)

---



Geister Medizintechnik GmbH  
Föhrenstraße 2  
78532 Tuttlingen / Germany  
Tel.: +49-7461-96624-0  
FAX: +49-7461-96624-22  
E-Mail: info@geister.com



Read usage instructions



Sterilized through irradiation



Not to be used in case package is damaged



Medical product higher class I and shortcut for Notified Body TÜV SÜD  
Production Service GmbH, München, Germany



Re-use is not allowed



Use by date



Batch code



Catalogue number

RxOnly

Prescription - U.S. federal law restricts this device to sale by or on the order  
of a physician or properly licensed practitioner

### Article-No.:

16-2070 15° Standard Knife blade&tip (pack of 6) sterile  
16-2075 Crescent Knife str. 2mm blade&tip (pack of 6) sterile

### Intended use

Microsurgical knives intended for cutting tissue and vessels.

### Sterility

Microsurgical knives delivered sterile. The package content is sterile, if the package is undamaged and closed. The device is sterilized through irradiation. The packaging should be opened only immediately prior to the procedure. After unpacking the device - general aseptic requirements must be complied. Microsurgical knives are single use products and it is not allowed to clean or re-sterilize them. The re-sterilization of the device (in packaging or after expiration date) is not allowed. The re-sterilization of the device may a risk to the patient (product failure). The

manufacturer accepts no liability for a re-sterilization of the device and for an application after the expiration date. This also applies for prematurely opened and a damaged packaging.

### **Storage**

The microsurgical knives are packaged in a blister pack and a sealed carton. The storage is in the unopened original packaging between 10° C and 30°C (50° F and 86° F) and a maximum humidity of 70%. Each package has a batch number and an expiration date. It is not allowed to use the device after the expiration date.

### **Attention**

According to the federal law of the United States, the sale of the device is permitted exclusively to doctors or other qualified persons.

### **Warnings and precautions:**

- Microsurgical interventions should be carried out by trained persons
- Device contact to body fluids require special disposal to prevent contamination
- Do not use devices with opened and a damaged packaging - the re-sterilization of the device may a risk to the patient (product failure)
- **Construction:** The microsurgical knives are sterile and single use products.

### **Information**

For further information or product demonstration, please contact - Geister Medizintechnik GmbH, Föhrenstr. 2, D-78532 Tuttlingen, Germany, Tel. +49(0)7461/966240, Fax +49(0)7461/9662422, E-Mail [info@geister.com](mailto:info@geister.com)