# EnligHTN™

Multi-Electrode Renal Denervation System RF Ablation Generator

#### **INDICATIONS FOR USE**

The EnligHTN RF ablation generator is intended to deliver RF energy to the EnligHTN renal artery ablation catheter.

#### **PRODUCT BENEFITS**

- The RF ablation generator is a component of the EnligHTN renal denervation system and facilitates control and monitoring of the ablation process
- Temperature-controlled system with four independent monitoring channels
- Pre-programmed default settings for use with the EnligHTN renal artery ablation catheter
- Proprietary algorithm to produce consistent, transmural renal artery ablations
- Automatic safety shutoff at ablation completion and in the event temperatures or impedance exceed the pre-set or safety limits
- Ability to selectively disable EnligHTN ablation electrodes if desired

#### **TECHNICAL SPECIFICATIONS**

- Default settings:
  - Power: 6 Watt
  - Temperature: 75°C
  - Impedance: 400 Ω
  - Time: 90 seconds per electrode
  - During ablation, real-time values for these parameters are displayed on each channel
  - Operating Mode: Temperature Controlled
  - Temperature Monitoring: 4-channel independent and simultaneous display
  - Automatic safety shutoff when:
    - The temperature exceeds 80°C,
    - The temperature exceeds the setting by more than 5°C for greater than 3 seconds, or
    - $\blacksquare$  The measured impedance falls below 50  $\Omega$  or exceeds 400  $\Omega$
- Dimensions: 362 mm x 153 mm x 317 mm (W x H x D without handle)
- Weight: 9.75 kg

### **SYSTEM COMPONENTS**

THE FOLLOWING SYSTEM COMPONENTS ARE PROVIDED WITH THE GENERATOR:

COMPONENT/REORDER NUMBER	DESCRIPTION
(1) EnligHTN Power Cable, reorder number ENL-PC-01	Medical-grade power supply approved by international safety agencies. This power supply can be used for various line voltages and frequencies without any modifications. Replacement power cables may be ordered separately.
(1) 1641 Connector Cable, reorder number ENL-CC-01	Reusable cables to connect the ablation catheter with the generator.  See the ENL-CC-01 Connector Cable Instructions for Use for resterilization instructions.  Replacement connector cables may be ordered separately.

THE FOLLOWING SYSTEM COMPONENTS ARE PROVIDED SEPARATELY:

COMPONENT/REORDER NUMBER	DESCRIPTION
EnligHTN Renal Artery Ablation Catheter, reorder number ENL-SM-01 or ENL-LG-01	Ablation catheter required for use with the EnligHTN RF ablation generator.  See the EnligHTN renal artery ablation catheter Instructions for Use for product details.
EnligHTN Guiding Catheter, reorder number ENL-8F55	Guiding catheter facilitates percutaneous introduction of the EnligHTN renal artery ablation catheter.
Patient Return Electrode, reorder number PREP-01	Patient return electrodes used with the EnligHTN renal denervation system must be IEC 60601-2-2 compliant.  Replacement patient return electrodes must be ordered separately.





## **Ordering Information**

EnligHTN Renal Denervation System		
Product Name		
EnligHTN RF Ablation Generator		
EnligHTN Renal Artery Ablation Catheter - large		
EnligHTN Renal Artery Ablation Catheter - small		
EnligHTN Guiding Catheter		
EnligHTN 1641 Connector Cable		
Patient Return Electrode		
EnligHTN Power Cable - replacement		

ATRIAL FIBRILLATION CARDIAC RHYTHM MANAGEMENT CARDIOVASCULAR NEUROMODULATION

**Global Headquarters** One St. Jude Medical Drive St. Paul, Minnesota 55117

+1 651 756 2000 +1 651 756 3301 Fax

St. Jude Medical Brasil Ltda.

Rua Itapeva, 538 5° ao 8° andar 01332-000 – São Paulo – SP Brazil

+55 11 5080 5400 +55 11 5080 5423 Fax

Cardiovascular Division 177 East County Road B St. Paul, Minnesota 55117

+1 651 756 4466 Fax

+1 651 756 4470

St. Jude Medical Japan Co., Ltd.

Shiodome City Center 15F 1-5-2, Higashi-Shinbashi Minato-ku Tokyo 105-7115 Japan +81 3 6255 6370

+81 3 6255 6371 Fax

SJM Coordination Center BVBA The Corporate Village Da Vincilaan 11 Box F1 1935 Zaventem, Belgium +32 2 774 68 11 +32 2 772 83 84 Fax

St. Jude Medical (Hong Kong) Ltd. Suite 1608, 16/F Exchange Tower 33 Wang Chiu Road Kowloon Bay, Kowloon Hong Kong SAR +852 2996 7688 +852 2956 0622 Fax

SJMenligHTN.com SJMprofessional.com





EnligHTN, ST. JUDE MEDICAL, the nine-squares symbol and MORE CONTROL. LESS RISK. are registered and unregistered trademarks and service marks of St. Jude Medical, Inc. and its related companies.

